

Cryolipolysis fat freeze Slimming Machine

User Guide



Please fill in water full before turning on the machine:(Distill water/normal clean water. 3L) exchange water once in 2 weeks.
connect the handle,push straight tighten and lock the side tips.

Steps of filling in water:

open “overflow”.

insert water funnel to”injection”.

fill in approximately 3L water.

when “Overflow” comes out water.it is full.

Press the injection blue hole frame to take out the water funnel.

Turning on machine:press the green button and it will light on.

Machine Installation.

Front and back:



Left and Right:



Contents

Operating Manual

1.1 General Structure

2.1 Key Technology

2.2 Main Function

2.3 User Interface

3.1 Points For Attention

4.1 Contraindication


4.2 Warranty

4.3 Cautions

4.4 Technical Data and packing list

1.1 General Structure

1. Cryolipolysis + Vacuum + Photon Blue ledLight

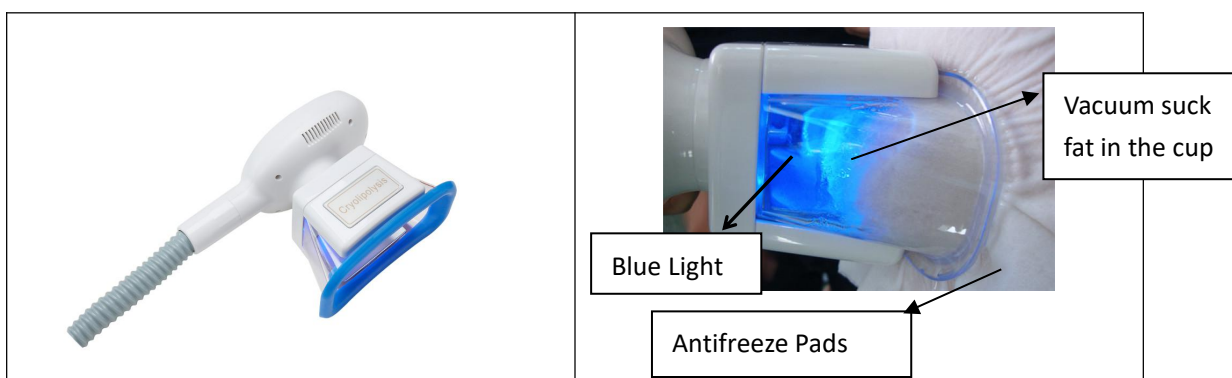

Big size vacuum cup with silicone frame.for abdomen.

Selection of different size handle for leg,butt,arm and chin.


2.1 Key Technology

Minus -14°C freezing vacuum sculptor/Vacuum suction fat in cup

2.2 Main Function

- 1.Body shaping,fat dissolving,repel the cellulite.
2. Lymphatic drainage,promoting metabolism&blood circulation.
- 3.Decrease the excessive water and prevent the fat from accumulation.
- 5.Fastest effect fat removal.
- 6.Complementing the collagen,skin rejuvenation.



2.3 User Interface

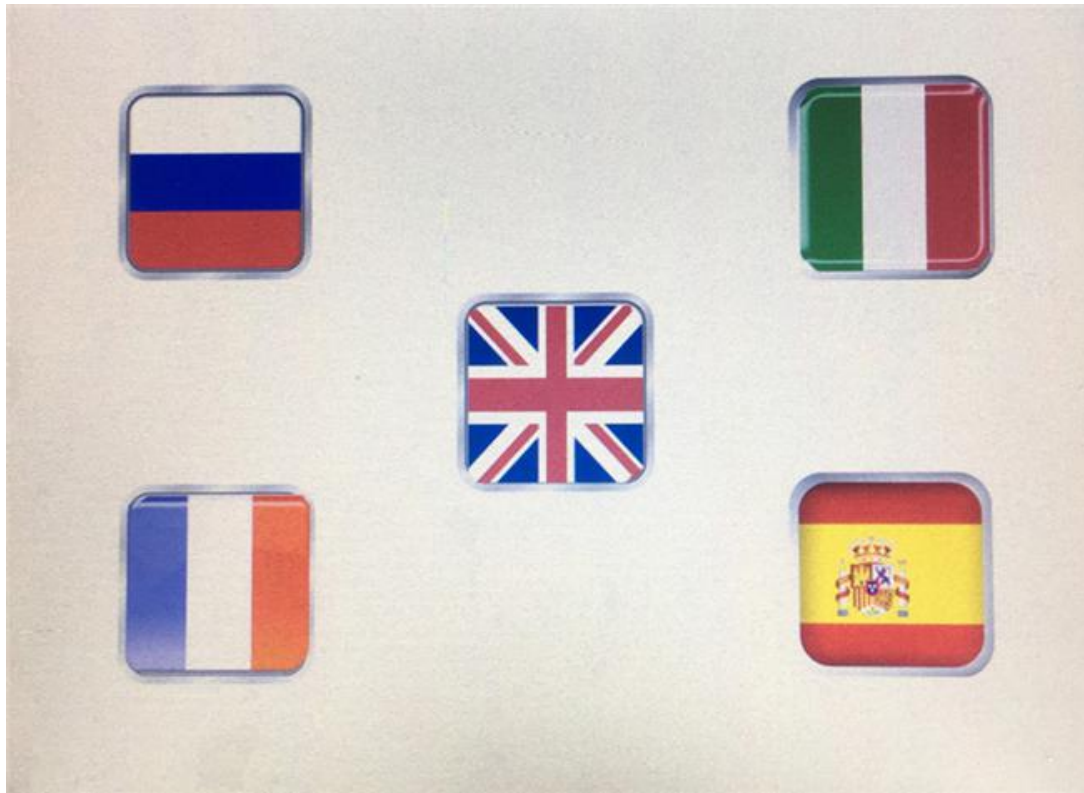

Select Language to enter in Software.

- ❖ The main menu interface for Cryolipolysis+vacuum and handpiece control display




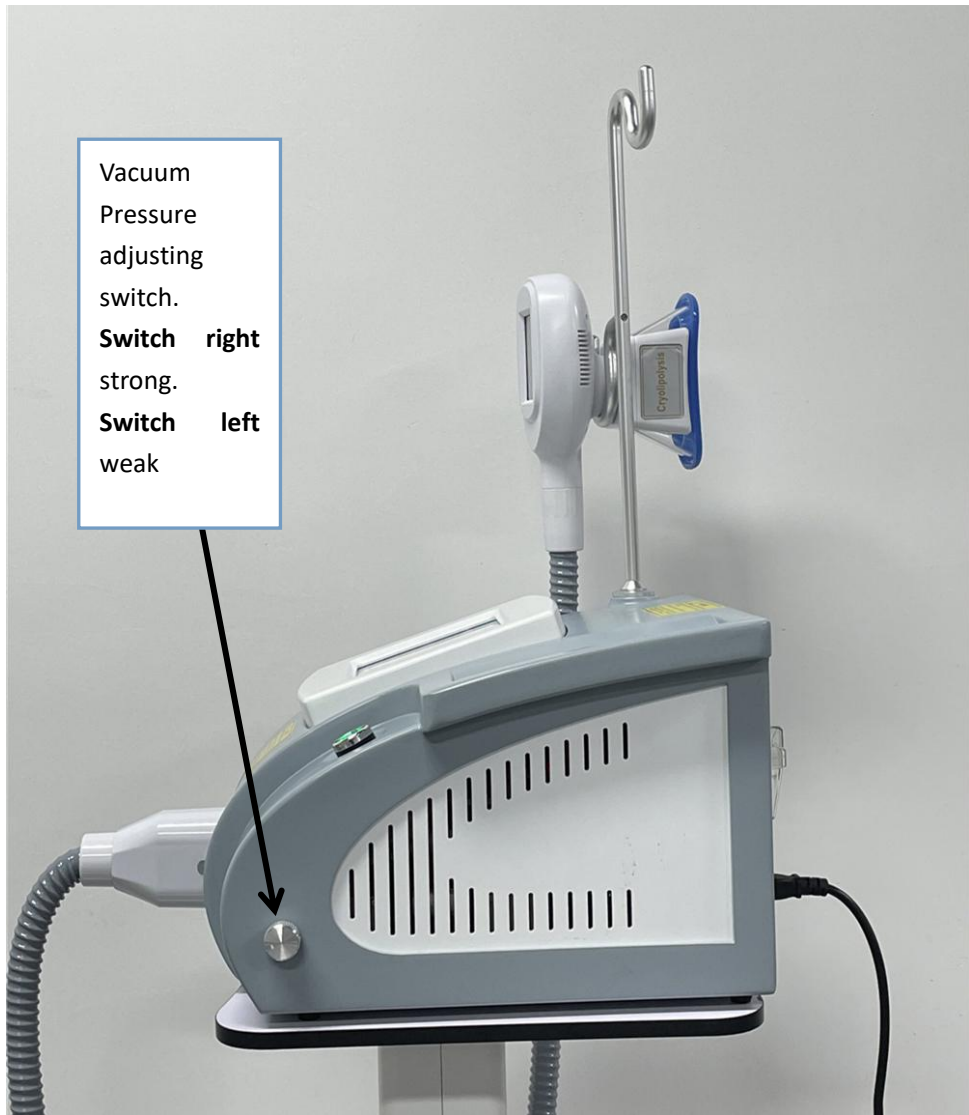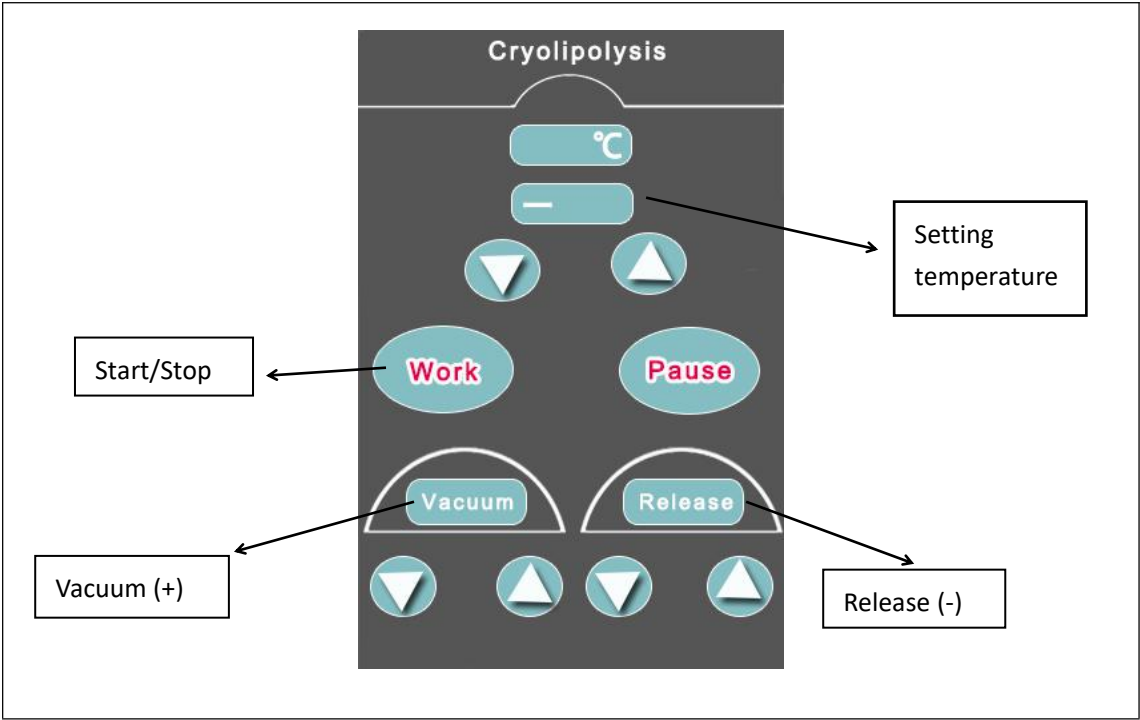

PS:(If you have selected chin handle)

Steps for enter in double chin operation screen:

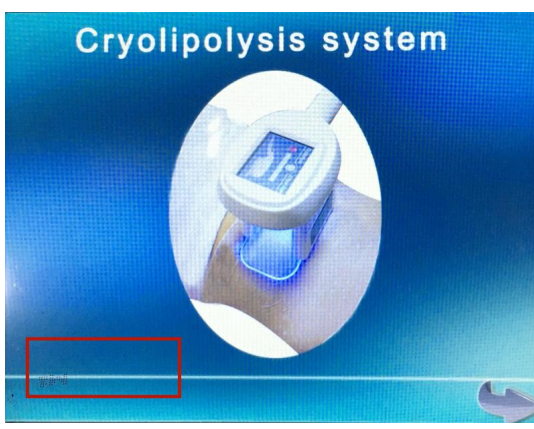

 <p>Click the frame 3 times</p>	 <p>Chin handle screen</p>
--	---

➤ **Cryolipolysis Treatment settings**

1. -5 degree freeze treatment time 20-30 minutes for each part,-10 degree treatment time 15-20 mins.-14 degree no more than 15 mins.to avoid frostbite.antifreeze membrane disposable for each client. Setting for vacuum 3S. release 1S.Machine body host screen 1 snow is -3 degree. Press handle screen “work” to start vacuum and handle.
2. Multi-sizes handles for different area treatment.each area treatment time 20-30 mins.move the handle step by step.
3. Take a another cooling procedure at the same position.you needs to wait at least 10 days. Wait for first procedure fat remove out of body and skin recover from Metabolism.
4. Cryolipolysis cool-sculpturing treatment is designed for those physically healthy people who have local fat deposits.It can target at the fat at a specific part, such as waist, abdomen, back, etc.

3.1 Points for attention

1. Read the user manual and machine body yellow stickers first before operation.
2. Use antifreeze pads on skin for treatment.
3. The patient cannot receive the treatment if he/she has following medical electronic equipment:
 - Implanted cardiac pacemakers or other implanted electronic devices;
 - Life-support artificial heart-lung machine or other equipment;
 - Portable ECG measurement device;

4.1 Contraindication

Please avoid the treatment from these people :

1. Pregnant women or women who may be pregnant.
2. Scarring, infection of the treatment area
3. History of keloid scarring
4. Use of medication known to induce photosensitivity
5. Fragile skin
6. Patients undergoing the surgical procedure without consulting with a surgeon
7. Patients with tumors
8. Patients with serious disease
9. Patients with low/high blood pressure

4.2 Warranty

1. Manufacturer warrants defects in materials and workmanship for a period of 1 year from the date of original purchase.
2. Manufacturer won't bear any responsibility:
 - A. Damage caused by accident, abuse, misuse or misapplication, it will be repaired with repair fee.
 - B. Damage caused by service performed by anyone who is not technician or qualified by manufacturer.
 - C. A product or part that has been modified without any permission of manufacturer.
 - D. machine is been opened without notice to manufacturer. disposable screw sticker is gone.

4.3 Technical Parameters

- 1.Host display: 8 "TFT color touch screen
- 2.Voltage : 220V/50Hz or 110V/50Hz
- 3.Power:>400W
- 4.Cooling liquid: distill water/clean water
- 5.Vacuum: 650mmHg
- 6.Pump flow rate: 60L/min
- 7.Vacuum pressure: 0-86Kpa
- 8.Freezing temperature: 0°C, -10°C, -14°C
- 9.**(big) body cryo handle size 17*10cm(standard)**

(Medium) Leg cryo handle size 14*5cm(selective to buy additional)

(Small) Chin freeze handle size 8*4cm (selective to buy additional)



Packing list:

- Machine Body with power wire 1 pcs
- Handle support 1 pcs
- Cooling handle 1 pcs/2 pcs/3 pcs
- Water funnel 1 pcs
- Vacuum filter 1 pcs
- User brochure 1 pcs